

Caries Detector



Dector® is a propylene glycol solution of green dye pre-loaded into **Skubes®**, small sponges able to absorb the necessary amount of solution. **Dector** is intended to assist with the visualization of carious dentin. It can be used to identify caries, root canals, and cracks in areas which are hard to observe. **Dector** is also used to avoid removal of sound dentin when excavating deep carious lesions.

DIRECTIONS

1. Properly isolate the carious tooth.
2. Clean and dry the carious tooth.
3. Dispense one or two **Skubes** into a dappen dish. When needed, pick up with cotton pliers and gently wipe over the dentin surfaces you want to check.
4. Wash the area after 10 seconds.
5. Remove the visibly stained carious dentin.
6. Repeat the detection and removal process a few more times until the prep no longer stains.
7. Proceed with a routine restoration.

STORAGE

Do not expose to temperatures exceeding 75° F (24° C).
Do not expose to direct sunlight.
Do not freeze.

LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical (800) 445-0345

Cerinate® Studios customer service (800) 872-8384

Domestic fax ordering and inquiry (800) 233-6628

International inquiry (805) 347-7990

International fax ordering and inquiry (805) 349-2607

Websites: www.lumineersdds.com
www.denmat.com

MSDS SHEETS AVAILABLE AT WWW.DENMAT.COM