

*Three minutes per day
for a whiter smile*



Doctor Instructions

1. Oral Exam

Examine the general condition of teeth, soft tissue health, and record the current tooth shade.

2. Mouthguard Fabrication

Follow the instructions below about mouthguard fabrication. Manufacture mouthguard in your office or send an impression to the Cerinate® Studios.

3. Instructions to Patient

Review "Patient Instructions" with your patient.

For 22% Concentration

The recommended bleaching regimen is 1 or 2 treatments per day for 1–2 hour durations for 2–3 weeks, or until desired results are achieved.

For 32% Concentration

The recommended bleaching regimen is 1 treatment per day for 3 minutes for 14 days, or until desired results are achieved.

4. Recall Appointment

Advise the patient that whitening is usually stable for several years. However, certain foods, beverages and tobacco products may cause gradual restaining. In these cases, a maintenance program may be necessary, such as touch-up whitening for 1–2 days as needed.

Schedule a recall appointment in 3 weeks.

Recommended Procedure for Fabricating a Sapphire Whitening Mouthguard

1. Take an impression of the arch(es) you want to treat. The impression should allow all areas of the clinical crown and at least 1/2 inch of gingival tissue surrounding the teeth to be accurately represented.
2. Immediately pour a fast-setting stone mixture into the impression.
3. After the model has set, trim the base to make it as thin as possible (1/16"-1/8"). A hole carefully drilled in the center of the base will improve the vacuum-forming process.

4. Place a thin layer of Paint-on Dental Dam on the stone model and light cure.
5. Place a 5"x5" sheet of .040 or .080 clear resin mouthguard material into the clear resin holder on the upper part of the vacuum former.
6. Set the stone model in the center of the perforated plate of the former and allow the resin sheet to heat until it droops 1" from the clear resin holder. Activate the vacuum switch and lower the resin sheet onto the model.
7. When the mouthguard resin has cooled, remove it and the model from the vacuum former. Remove the resin from the model and trim (scallop) the labial and lingual surfaces of the mouthguard to the gingival margins.
8. Polish the scalloped margins on a lathe using a cloth wheel only. Edge transparency can be recovered by the careful use of an alcohol torch. Wash and clean the mouthguard thoroughly.

Supplemental Products:

Sapphire Professional Home Whitening Kit 22% Fruit034348145
Sapphire Professional Home Whitening Kit 22% Mint034338145
Sapphire Professional Home Whitening Kit 32% Fruit034238145
Sapphire Professional Home Whitening Kit 32% Mint034138145
Sapphire Professional Home Whitening Kit 22% Fruit Refill034348425
Sapphire Professional Home Whitening Kit 22% Mint Refill034338425
Sapphire Professional Home Whitening Kit 32% Fruit Refill034238425
Sapphire Professional Home Whitening Kit 32% Mint Refill034138425
Resin Sheets 25 pc00738
Sapphire Professional Chairside with Aftercare Whitening Kit035258455
Sapphire Plasma Arc Curing and Whitening Light033958000
Sapphire Whitening Crystal030395250
Sapphire Brilliance Toothpaste 1oz 30pc30600500
Sapphire Brilliance Toothpaste 4.5 oz 12 pc35000300
Sapphire Polar Ice Toothpaste 4.5 oz 12pc30700300
Sapphire Sensitive Toothpaste 4.5 oz 12pc30800300
Sapphire Deep Stain Toothpaste 4.5 oz 12pc30900300
Sapphire Oral Care 12 pc Mix Pack (3 of each flavor)31100300

Warnings:

- Keep out of reach of children.
- Sapphire Professional Home Whitening System is supplied in a pre-filled syringe. This material is not intended for injection.
- Sapphire Professional Home Whitening System is not intended for patients who are pregnant or nursing.
- Sapphire Professional Home Whitening System should not be used by anyone with known allergy or sensitivity to peroxides.
- Patients with exposed dentin should be assessed for suitability to whitening prior to being given this product.
- Patient expectations should be discussed before treatment. Some teeth may not attain desired results such as existing crowns and bridges, and heavily restored teeth.
- Esthetic restorative procedures should be delayed until at least 2 weeks after whitening to ensure a good color match.

Questions?

Contact your account representative at 800-445-0345.

752383231W

LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical. (800) 445-0345

Cerinate® Studios customer service. (800) 872-8384

Domestic fax ordering and inquiry. (805) 349-2600

International inquiry (805) 347-7990

International fax ordering and inquiry (805) 349-2607

Websites: www.lumineersdds.com
www.denmat.com

MSDS SHEETS AVAILABLE AT www.denmat.com