

## Professional Home Whitening System

### Easy Patient Instructions for a Whiter, Brighter Smile

1. Floss teeth and then brush gently.
2. Place a thin line of whitening gel into your custom-designed whitening trays as directed by your dental professional. Each tooth impression in the mouthguard should contain an adequate amount of gel. Try to avoid dispensing an excessive amount into the mouthguard. (If you are sensitive to peroxide, before inserting the trays, apply a thin amount of petroleum jelly to the gums adjacent to the teeth that will be whitened.)
3. Insert the whitening trays and allow any excess material to exude. Wipe away excess.

#### 4. **22% Concentration**

The recommended bleaching procedure for 22% is 1 or 2 treatments per day for 1–2 hour durations for up to 14 days or until you achieve desired results.

#### **32% Concentration**

**The recommended bleaching procedure for 32% is one 3-minute treatment per day for 14 days, or until you achieve desired results.**

5. Clean mouthguard thoroughly after each use and then brush your teeth again.
6. Rinse with alcohol-free mouthwash or warm water.
7. Schedule recall appointments with your dental office as instructed.
8. When you are ready for refills, contact your dental professional for additional Sapphire Professional Home Whitening Gel.

#### **Warnings:**

Keep out of the reach of children. Not for injection.

#### **Storage:**

Store at or below 75° F (23° C)

#### **LIMITED WARRANTY**

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

#### **TELEPHONE INQUIRY SERVICE**

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical . . . . . (800) 445-0345

Cerinate® Studios customer service . . . . . (800) 872-8384

Domestic fax ordering and inquiry . . . . . (800) 233-6628

International inquiry . . . . . (805) 347-7990

International fax ordering and inquiry . . . . . (805) 349-2607

Websites: [www.lumineersdds.com](http://www.lumineersdds.com)  
[www.denmat.com](http://www.denmat.com)

**MSDS SHEETS AVAILABLE AT [WWW.DENMAT.COM](http://WWW.DENMAT.COM)**