

DENMAT CUSTOMER SUPPORT

U.S. and Canada Customer Service..... 800-433-6628
U.S. and Canada Fax Orders and Inquiries..... 800-233-6628
International Inquiries..... 805-347-7990
International Fax Orders and Inquiries: 805-347-7983
Website..... denmat.com

SAFETY DATA SHEETS AVAILABLE AT denmat.com

Gloss-N-Seal® Light-Cure Sealant



Symbols Glossary available at:
denmat.com/symbols



Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



INSTRUCTIONS

Gloss-N-Seal® is a light-cure, low-viscosity surface sealant with exceptional abrasion resistance. It cures in a very thin, glazed layer with a minimal oxygen-inhibited layer. Gloss-N-Seal penetrates and seals, thus reducing composite wear, marginal staining, and breakdown. Applications include: sealing composite restorations, coating provisionals for instant glazed finishes, and resurfacing old composite restorations.

CAUTION: Wear protective gloves while using this product.

CAUTION: Wear eye protection while using this product.

CAUTION: Gloss-N-Seal® has not been studied in children, pregnant or breast-feeding women.

1. Following occlusal adjustment, rinse and air-dry the surface (tooth as well as composite).
2. For maximum bond strength apply one layer of Tenure Uni-Bond to the entire restoration/tooth surface, and gently air thin.
3. Apply one thin coat of Gloss-N-Seal, and light-cure for the final seal and finish, starting with:
 - With Sapphire PAC lights (all models) start with 5 second exposures.
 - With Flashlite LED lights (all models) start with 10 second exposures.
 - For all other curing lights; halogen, LED and other refer to the manufacturer instructions. A minimum of 10-30 seconds is recommended.

STORAGE

Do not expose to temperatures exceeding 77° F (25° C).

Do not expose to direct sunlight.

Cold storage is recommended in tropical climates.

Recommended temperature when used is 70° F-75° F (21°-24° C).

RELATED DENMAT PRODUCTS

Description Number	Product
Tenure® Multi-Purpose Bonding System.....	031146000
Tenure Quik® with Fluoride Kit.....	031145700
Core-Post Value Kit	033637000
Virtuoso® Flowable Custom Kit	030381800
Virtuoso® Universal Kit	030381950
Bio-Cap® Adhesive Liner Kit	030451110
Geristore® Syringeable Kit	031457500

LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.