

LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

DENMAT CUSTOMER SUPPORT

U.S. and Canada 800-433-6628
U.S. and Canada Fax Orders and Inquiries..... 800-233-6628
International Inquiries 805-347-7990
International Fax Orders and Inquiries: 805-347-7983
Website denmat.com



Tenure Quik with Fluoride
Single-Bottle Bonding Agent
with Fluoride, Refill



REF 031145300

Do Not Refrigerate

 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA



EC REP MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

CE 0044

Symbols Glossary available at: denmat.com/symbols

©2019 Den-Mat Holdings, LLC. All rights reserved. 751453070 R-9 01/19CM

CAUTION: Wear protective gloves while using this product.

CAUTION: Wear eye protection while using this product.

CAUTION: This product has not been studied in children, pregnant or breast-feeding women.

DIRECTIONS

Instructions for Root/Dentin Desensitization

1. Clean the sensitive area with Porcelain Polishing Paste.
2. Rinse with warm water and blot dry with a cotton pellet.
3. Apply multiple coats (4-6) of Tenure Quik with Fluoride to the surface. Air-dry with a syringe for several seconds to remove the solvent. Light-cure Tenure Quik with Fluoride as directed in Step 4 of "Instructions for Bonding to Dentin."
4. Repeat Step 3.
5. For longer lasting desensitization, apply a thin layer of restorative material (for example, Virtuoso® Flowable) on top of the Tenure Quik With Fluoride.

Instructions for Bonding to Dentin

1. Etch the tooth surface with 37% phosphoric acid etchant for 15 seconds. Thoroughly rinse and gently blot or air-dry the tooth surface. (Fig. 1)
2. Completely immerse the brush tip in Tenure Quik® with Fluoride. Using a fully saturated brush tip, apply three consecutive coats to the prepared tooth. Allow Tenure Quik with Fluoride to sit on the tooth for 15 seconds. (Fig. 2)
3. Remove the solvent from the Tenure Quik with Fluoride by directing a gentle stream of dry, oil-free air from the air syringe. The force of the air should be strong enough to extract the adhesive's solvent, but not so forceful as to push the adhesive from the preparation. If the surface is not glossy after drying, reapply Tenure Quik with Fluoride and gently re-dry.
4. Light cure Tenure Quik with Fluoride (Fig. 3), starting with:
 - Sapphire PAC lights (all models) start with 5 second exposures.
 - Flashlite LED lights (all models) start with 10 second exposures.
 - For all other curing lights; halogen, LED and other refer to the manufacturer instructions. A minimum of 10-30 seconds is recommended.

Apply the restorative material as directed by the manufacturer.

STORAGE

Do not freeze.

Do not refrigerate.

Do not expose to temperatures exceeding 77° F (25° C).

Do not expose to direct sunlight.

Recommended temperature when used is 70° – 75° F (21° – 24° C).

Cool storage is recommended in tropical climates.

Bring to room temperature prior to use.

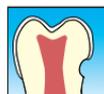


Fig. 1

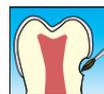


Fig. 2



Fig. 3