

Visar Seal[®]

Light-Cure Resin

Store in a cool, dry location

Visar Seal is a light-cure, clear resin liner that can be used as a glazing agent, pit and fissure and composite sealant, and in many bonding procedures as a wetting agent.

DIRECTIONS

1. Remove the cap (do not discard) and dispense into a dappen dish.
2. With a brush apply Visar Seal to the prepared surface (etched enamel or cleaned and dried composite).
3. Gently blow with air to spread the Visar Seal to a thin, even film. It is important that you avoid pooling the mixture.
4. Complete one of the following options:

Option 1: Polymerize immediately:

- Sapphire[®] PAC lights (all models) start with 5 second exposures.
- Flashlite[®] LED lights (all models) start with 10 second exposures.
- For all other curing lights; halogen, LED and other refer to the manufacturer instructions. A minimum of 10-30 seconds is recommended.

Option 2: Place the restorative directly over the wet, thin film of Visar Seal.

5. Replace the cap onto the Visar Seal Bottle.

ADDITIONAL USES

1. Lubricates plastic or metal instruments—preventing composite material from sticking to instruments during mixing or placement.
2. Reduces viscosities of thicker, more viscous composites—improves composite flow.
3. Condense posterior composite restorations without pulling—use a Skube[®] saturated with Visar Seal.

CAUTION: Wear protective gloves while using this product.

CAUTION: Wear eye protection while using this product.

CAUTION: This product has not been studied in children, pregnant or breast-feeding women.

STORAGE

Do not expose to temperatures exceeding 77° F (25° C).

Do not expose to direct sunlight.

Do not freeze.

Cold storage is not required.


RELATED PRODUCTS AVAILABLE FROM DEN-MAT

Description	Kit Number
Skubes® 900 pc	0065
Visar Seal® 4g	3806
Visar Seal® 10g	3810
Sapphire® Plus Plasma Arc Curing Light	033968000

SAFETY DATA SHEETS AVAILABLE AT www.denmat.com

Symbols Glossary available at:
www.denmat.com/symbols



 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

