


Mounting Option: To “lock” your Sapphire Light on the rolling stand, place the upper clamp just above your new Sapphire Light and tighten. The clamp touches or comes close to the top of the light and makes it impossible to tip out or remove. **Use this mounting option if you do not plan to use the light in a portable manner.**

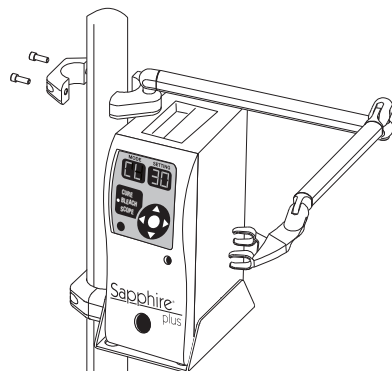

LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical (800) 433-6628
Denmat Lab Studio Customer Service (800) 872-8384
International inquiry (805) 347-7990

MSDS SHEETS AVAILABLE AT denmat.com



©2018 Den-Mat® Holdings, LLC. Lompoc, CA 93436 705941090 R:2 07/18CM

Fast. Convenient. Effective.

For beautiful, white smiles when your patients want immediate satisfaction, Lumibrite® Chairside Whitening is the answer. Used with DenMat's Sapphire® light, Lumibrite® Chairside Whitening achieves results of up to 12 shades. That's sure to turn some heads.

For more information call 800-433-6628 or visit denmat.com

Sapphire®





STAND WITH WHITENING CRYSTAL ARM




We recommend testing your new Whitening Crystal Arm with an assistant before using it with a patient. This will help you become familiar with the movements of the Whitening Crystal Arm.

Use a combination of chair adjustments with articulating arm adjustments when positioning your patient next to the Whitening Crystal Arm.


Contents

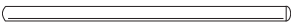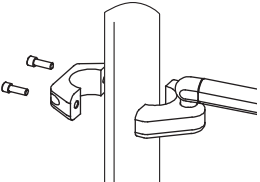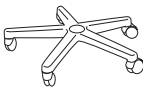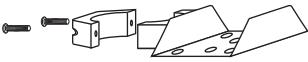Post 	Whitening Crystal Arm 
Caster Base 	
Sapphire Mounting Bracket 	4 Hex Keys 1/4" Tighten Post Clamps 10 mm Attach Post to Caster Base 5/32" For Arm Elbow Adjustment 3/16" For Arm Wrist Adjustment


Note: The **Sapphire Light** Mounting Bracket is preassembled eighteen inches below the top of the post. When adjusting this bracket use the 1/4" hex key provided.

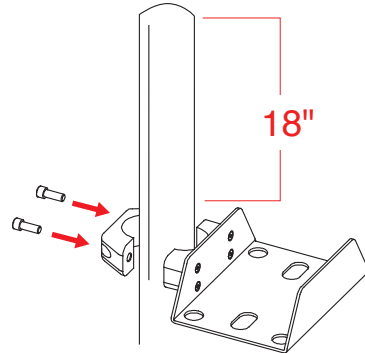


- Using your fingers or the 10 mm hex key, remove the hex bolt and base washer from the bottom of the post.

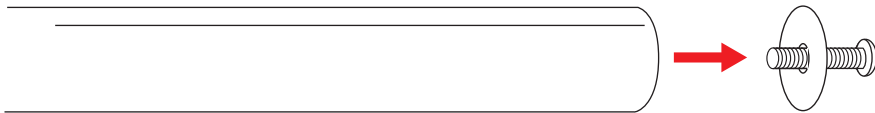

- Firmly place the post into the caster base.

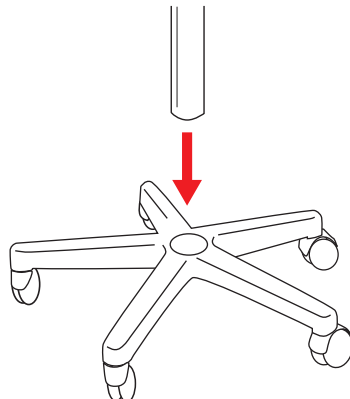


- Place the combined unit on its side and position the hex bolt and base washer into the slot and tighten using the 10 mm hex key.

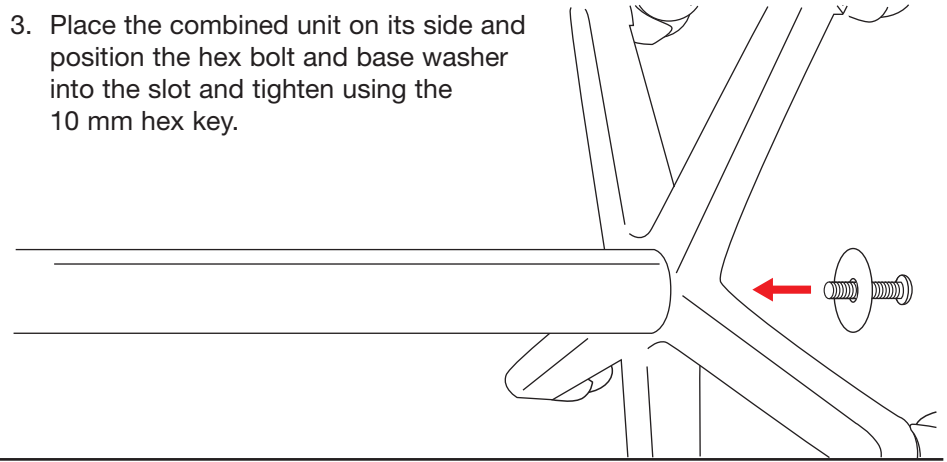


- Position the unit upright. With the help of an assistant, place the **Whitening Crystal Arm** three inches below the top of the post and tighten with the 1/4" hex key.

NOTE: The ideal positioning of the arm is directly above the Sapphire Light Mounting Bracket.

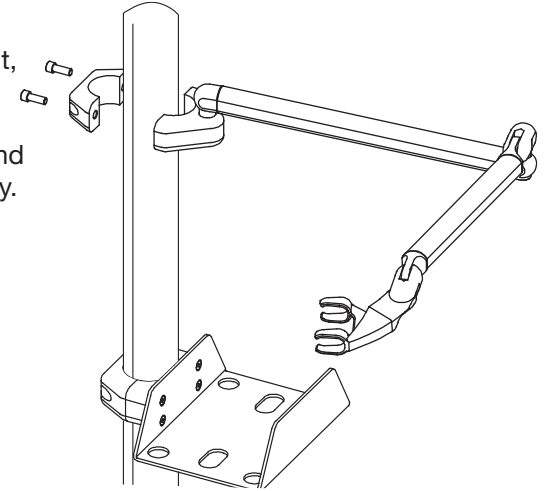

- The torque on the joints of the **Whitening Crystal Arm** may be tightened or loosened with the 5/32" and 3/16" hex keys.

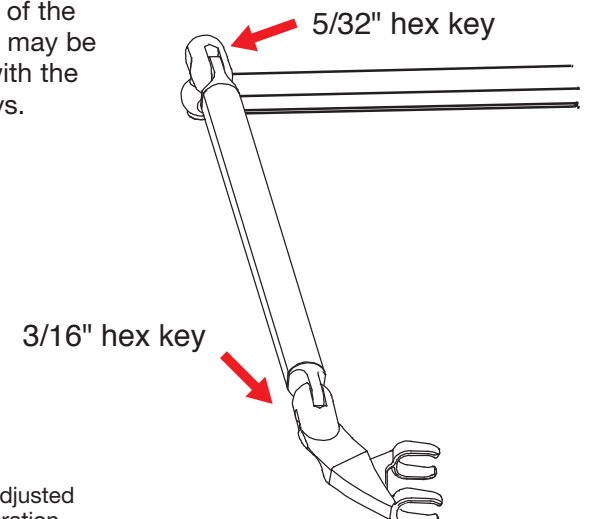

NOTE: These joints are preadjusted at the factory for proper operation.