calcium phosphate  
desensitizing gel

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Zen CP+ desensitizing gel

## DESCRIPTION

Zen CP+ is a topical oral gel dentifrice exclusively with Calcium Magnesium Phosphate (CAPOSAL®) and Potassium Nitrate used in the protection against tooth sensitivity.

Calcium Magnesium Phosphate provides fast and complete occlusion of dentin tubules due to higher bioactivity on the dentin surface and within the tubules of the teeth. Zen CP+ also contains Potassium Nitrate which is known to increase the protection against sensitivity of the teeth by nerve depolarization.

## RECOMMENDED INDICATIONS

To be used for cleaning and polishing procedures as part of a professionally-administered prophylaxis treatment.

Secondarily, Zen CP+ can be used for the management of tooth sensitivity, after scaling and root planing, and for tooth whitening post-care.

## CONTRAINDICATIONS

None

## WARNINGS

- If pain from sensitive teeth worsens, stop using the gel and contact a medical professional
- If troublesome side effects occur, stop using the gel and contact a medical professional
- Avoid swallowing the gel

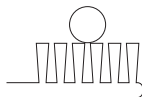
- Not intended for use by individuals under the age of 18 or women who are pregnant or breast feeding
- Keep the gel out of reach of children
- May experience minor tingling sensation of the gums after use

## DIRECTIONS FOR USE

For adults 18 years of age or older:

1. Unscrew cap and remove foil seal before use.
2. Brush teeth with standard toothpaste before use of Zen CP+.
3. Apply a dot of Zen CP+ the size of a pea (approx. 7mm) to an extra soft-bristled toothbrush or finger.
4. Apply to sensitive tooth with gentle motions and light pressure to affected areas for approximately 30 seconds.
5. Repeat process for each sensitive tooth.
6. Spit out excess gel if needed. Avoid rinsing your mouth with water.
7. Avoid eating or drinking for 45 minutes after application.
8. Use Zen CP+ once in the morning and once in the evening for 3 weeks.

(the size of a pea/approx. 7 mm):

**Note:** It is recommended that the gel is not used for longer than 3 weeks.

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All rights reserved.