

Symbols Glossary available at:  
[denmat.com/symbols](http://denmat.com/symbols)



Den-Mat Holdings, LLC  
1017 W. Central Avenue  
Lompoc, CA 93436 USA



MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany



## Multi-Purpose Bonding System



Tenure is a 4th generation bonding agent for all intra-oral surfaces which is highly effective, easy-to-use and eliminates sensitivity upon contact. The simple procedure creates bonds to dentin that approximate the strength of bonds to etched enamel.

**CAUTION:** Wear protective gloves while using this product. 

**CAUTION:** Wear eye protection while using this product.

**CAUTION:** Tenure® has not been studied in children, pregnant or breast-feeding women. 

## INSTRUCTIONS

### Enamel and Dentin

1. Remove all caries and/or establish margins of restoration.
2. Apply Etch 'N' Seal® to the enamel and dentin for 10-15 seconds.
3. Rinse thoroughly and dry.
4. Mix equal drops of Tenure A and Tenure B into dappen dish.
5. Apply 4 to 6 coats to the tooth surfaces with brush or until glossy in appearance.
6. Allow to fully evaporate, 10-15 seconds.
7. Optional: Apply one thin coat of Tenure S to the surface and light cure.\*
8. Proceed with restoration of choice.

### Old Composite/Resin

Roughen or sandblast surface. Rinse and dry.  
Proceed with steps 4-8.

### Metals

Roughen or sandblast surface. Rinse and dry.  
Proceed with steps 4-7.  
Apply GoldLink®, Geristore®, or Infinity® and restoration of choice.

\*Tenure S Bond Enhancer is recommended for wetting purposes and acts as a final sealer.  
Tenure S Filled is recommended for direct composite restorations as it is more viscous.

## Contraindications

Patients allergic or hypersensitive to methacrylate monomers.

## LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

## STORAGE

**Do not** freeze.

**Do not** refrigerate.

**Do not** expose to temperatures exceeding 77° F (25° C).

**Do not** expose to direct sunlight.

Recommended temperature when used is 70° – 75° F (21° – 24° C).

Cool storage is recommended in tropical climates.

Bring to room temperature prior to use.

## DENMAT CUSTOMER SUPPORT

U.S. and Canada.....	800-433-6628
U.S. and Canada Fax Orders and Inquiries.....	800-233-6628
International Inquiries.....	805-347-7990
International Fax Orders and Inquiries:.....	805-347-7983
Website.....	denmat.com

**SAFETY DATA SHEETS AVAILABLE AT [denmat.com](http://denmat.com)**