

# Laser Eyewear Instruction Manual



## Indications for Use

- Failure to follow instructions may result in serious eye injury. Read all instructions before using this product.
- Always be sure that the wavelength marked on your laser eyewear include the wavelengths of the laser in use.
- All persons in the room where the laser is in operation MUST wear laser protective eyewear.
- Be sure that your laser eyewear fits correctly and securely.
- Check laser eyewear before each use for damage. This includes lens which are cracked, pitted, scratched or damaged in any manner.
- Store laser eyewear in the case that has been provided with the laser eyewear.
- All laser eyewear is clearly marked with wavelength (nm) and absorption (OD) Optical Density. Absorption curves available upon request. Color recognition (of warning lights, etc.) may be impaired by use of tinted filters.

## Danger

- Always double check that the wavelengths marked on the laser eyewear includes the wavelengths of the laser being used.
- Laser eyewear are made only for use with the laser wavelengths that are marked on the lens.
- Avoid direct, indirect or reflected exposure to laser energy.
- Wear laser eyewear whenever the laser in in use.

## Warning

- NEVER LOOK DIRECTLY INTO THE PATH OF A LASER BEAM. Laser eyewear offers protection against accidental exposure to stray or diffused reflection of laser beam energy for a maximum exposure of 10 seconds.
- Do not modify this product without the prior approval of the manufacturer.
- Do not clean laser eyewear with abrasive cloths or liquids.
- Laser eyewear protects only the wearer. All personnel within the work area should wear appropriate eye protection against possible exposure to reflected beam energy.

## Cleaning Instructions

- Use enclosed cleaning cloth. Wash laser eyewear in mild soap and water. Rinse thoroughly with clean water. Dry with a clean, soft cloth.
- Do not use abrasive chemicals or cleaning compounds such as alcohol as this may result in damage to the laser eyewear.

## 6 Month Limited Warranty

- The product is protected against defects in materials and workmanship for 6 months from the date of original purchase.
- If the product proves to be defective in materials or workmanship during this period, it will be replaced free of charge.
- This warranty does not apply to damage resulting from misuse, accidents or alterations to the product, or to damages incurred in transit.
- This warranty does not apply to cords, cloths, or cases.

