

ub Ultra-Bond®


Resin Cement Try-In Paste, Supreme White

U.S. Patent: 7,175,700

dm
DenMat®



Ultra-Bond Plus Try-In Paste is a non-setting resin based paste which allows you to preview shades before cementation of porcelain veneers. The Try-In Paste simulation allows you to see what the veneers will look like once bonded in place with Ultra-Bond Plus, fully cured.

 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Symbols Glossary available at: www.denmat.com/symbols

CAUTION: Wear eye protection while using this product.

CAUTION: Wear protective gloves while using this product.

CAUTION: UltraBond products contain methacrylate monomers which have been known to cause allergic reaction in individuals sensitive to these ingredients.

CAUTION: Ultra-Bond has not been studied in children, Pregnant or Breast-feeding women.

TRY-IN

1. Preparation of Lumineers®

- After the patient is seated, treat the Lumineers with Porcelain Conditioner for 30 seconds, rinse and dry well.
- Apply Cerinate Prime® with a disposable brush to the Lumineers for 30 seconds and blow thin. (DO NOT use cotton pellet or sponge.)
- Place Lumineers back into the case box in their proper slots.
- Close the lid on case box to shield from light.

2. Preparation of Tooth Surfaces

- Clean the teeth with a slurry of Porcelain Polishing Paste.
- Etch teeth for 20 seconds with Etch 'N' Seal®, rinse and dry.
- Mix together equal amounts of Tenure® A and B. Apply 3-5 coats (or until glossy) to each tooth, then air-dry the tooth surfaces.

3. Try-In of Lumineers

- Syringe the Ultra-Bond® Plus Try-In Paste into each of the Lumineers.

Note: If using light-cured TetraPaque® to modify Try-In Paste, protect Lumineers from exposure to intense light.

- Gently insert the Lumineers one at a time and use a brush to clean up the excess Try-In Paste.

4. Show Patient

- Hold a mirror approximately 16 inches away from the patient's face and let them examine their smile.

- If the shade is not what you or the patient wants, remove the Try-In Paste from the Lumineers and tooth surface with a Skubes® or a clean, dry brush. Reapply a new shade of Try-In Paste and check the shade again.

5. To Remove Lumineers After Try-In

- Use a Schure 349 instrument or cotton pliers and gently tease the Lumineers off the tooth surfaces.
- Remove the Try-In Paste from the Lumineers and tooth surfaces with a clean brush. DO NOT USE Tenure S. Only use Tenure S to clean up excess once final Try-In Paste has been selected.
- Gently thin the Tenure S with air. A thin film will be left on both the tooth and porcelain surfaces.

Note: There is no need to reapply Porcelain Conditioner or Cerinate Prime if you are cementing directly after try-in.

For application of Lumineers, see the Ultra-Bond Plus cement kit instructions.

STORAGE

Do not expose to temperatures exceeding 75° F (24° C).

Do not freeze.

RELATED PRODUCTS AVAILABLE FROM DENMAT

Description	Kit Number
Ultra-Bond Plus Multi-Shade.....	032599750
Tenure Uni-Bond® with Gloss-N-Seal®.....	030411900
Tenure MPB System Kit.....	031146000
Porcelain Conditioner.....	1002AB
Cerinate Prime.....	1398B
Tenure S Dab-Eze	031145201
Sapphire® Plus Plasma Arc Curing Light.....	033968000

SAFETY DATA SHEETS AVAILABLE AT www.denmat.com