

TELEPHONE INQUIRY SERVICE


Technical consultation is available by calling toll-free in the U.S.A. and Canada

Ordering, Product Technical Support	(800) 433-6628
Customer Service	(800) 433-6628
Customer Service Lab Consultant	(800) 872-8384 x5520
Domestic Fax Ordering and Inquiry	(800) 233-6628
International Inquiry	(805) 347-7990
International Fax Ordering and Inquiry	(805) 349-2607
Website	denmat.com

SAFETY DATA SHEETS AVAILABLE AT denmat.com

Symbols Glossary available at: denmat.com/symbols



 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA



MDSS GmbH Schiffgraben
41
30175 Hannover, Germany

©2020 Den-Mat Holdings, LLC. All rights reserved. 750958150 R7 01/20CM

Sapphire[®]

WHITENING CRYSTAL

US Patent: 6,733,290



Instructions For Use


DenMat

REF 030395250



The Sapphire® Whitening Crystal is for dual-arch bleaching and is designed for use with the Sapphire Plus Curing Light.

INSTRUCTIONS

Gently pull the curing tip off the Sapphire Plus Light. Insert the Sapphire Whitening Crystal into the end of the light guide, making sure that it is completely seated before proceeding.

Follow *Whitening Instructions* found in the Sapphire Plus Instructions For Use.

CLEANING THE WHITENING CRYSTAL — DO NOT AUTOCLAVE

Clean with a mild, non-abrasive liquid soap and thoroughly dry all surfaces.

CAUTION: Wear protective gloves while using this product.



CAUTION: Wear eye protection while using this product.



CAUTION: This product has not been studied in children, pregnant or breast-feeding women.

RELATED PRODUCTS AVAILABLE FROM DEN-MAT

Description	Kit Number
LumiBrite® Chairside Whitening System	035359155
LumiSmile® In-Office Whitening Kit - 25%	WHT0025
LumiSmile White In-Office - 25% with Take Home - 22% Combo	WHT0030
LumiBrite Desensitizing Gel	034159555
Paint-On Dental Dam Kit - Blue	404210
Paint-On Dental Dam Syringe - 3ml Blue	404201

LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.