Infinity SE Syringeable

Self-Etch, Dual-Cure Resin-Ionomer Cement

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©2019 Den-Mat Holdings, LLC. All rights reserved.
750968332 R:4 04/19CM

Symbols Glossary available at:
denmat.com/symbols

Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA

Infinity SE Syringeable is a multi-purpose, self-etch, dual-cure resin ionomer cement that is insoluble, releases fluoride, and is radiopaque. Infinity SE Syringeable is recommended for all-porcelain, zirconium, metal and ceramo-metal crowns and bridges, inlays and onlays, posts, and orthodontic bands and brackets.

Refrigerate for Maximum Shelf Life

- CAUTION:** Wear protective gloves while using this product.
CAUTION: Wear eye protection while using this product.
CAUTION: This product has not been studied in children, pregnant or breast-feeding women.

DIRECTIONS

- Clean the tooth surfaces thoroughly.
- Prepare the selected surface as indicated.
 - ENAMEL:** Infinity SE Syringeable is self-adhesive. For maximum retention when bonding to enamel with minimal bonding surface area, we recommend using with the Tenure® Multi-Purpose Bonding System.
 - DENTIN:** Infinity SE Syringeable is self-adhesive. For maximum retention of restorations with minimal bonding surface area, or restorations without mechanical retention, we recommend bonding with the Tenure Multi-Purpose Bonding System.
 - COMPOSITE/ METAL/PORCELAIN/ZIRCONIUM:** To re-seat old restorations, roughen the surface with a diamond bur or by sandblasting. Thoroughly wash and air-dry the surface. For all other restorations, no further preparation is required, other than the preparation provided by the laboratory.
- Prepare the auto-mix syringe:
 - Align the straight edge of the auto-mixing housing with the syringe flange.
 - Push the mixing tip onto the syringe and turn 90° clockwise until it stops.
 - Attach the intraoral tip firmly.

Important: You must extrude a pea-sized amount of material after placing the mixing tip on the syringe and discard this material. This is critical to ensure that Infinity SE Syringeable sets properly. Repeat this each time a mixing tip is placed on the syringe.

- Dispense Infinity SE onto the restoration, or for inlays directly onto the tooth.
- Seat the restoration.

Allow the restoration to set for 4-5 minutes, then remove any excess material. Or, firmly press down and hold the restoration while you use a curing light, briefly light-cure the excess material until it reaches gel consistency for easy removal.
- Remove any excess Infinity SE with an instrument or floss.
- Ask the patient to bite and hold for another 5 minutes before you dismiss them.

Note: For porcelain restorations and composite resin indirect restorations, Infinity SE can be light-cured with the Sapphire® Plus PAC Light or the Flashlite® LED curing light. We recommend that you use the curing rings provided to determine the proper curing time for your curing light.

Due to variations in the performance characteristics of light curing units, ALWAYS bench test restorative materials before use in vivo. Curing test rings are provided for this purpose.

- Fill the 2 mm deep well of the test ring and level material.
- Position the light transmitting element perpendicular to and approximately 2 mm-5 mm above the top surface of the ring.
 - With Sapphire PAC lights (all models) start with 5 second exposures.
 - With Flashlite LED lights (all models) start with 10 second exposures.
 - For all other curing lights; halogen, LED and other, refer to the manufacturer instructions. A minimum of 10-30 seconds is recommended.
- Use a dental probe to scrape test the hardness of the top and bottom surfaces. The bottom surface should be as hard as the top surface.
- If the bottom surface is not completely cured repeat steps (b) to (c). Repeat until the bottom surface is completely cured.
- Maintain a log including material, shade and associated curing exposure time. Use the log to monitor system performance.

Note: If a cavity preparation is deep, curing exposure times must also be increased due to beam divergence and angular placement of the light transmitting element to the restoration. An incremental filling technique is recommended and each increment should be fully cured prior to applying additional layers.

General guidelines for curing light unit exposure times.

See manufacturer's instructions. ALWAYS bench test restorative materials before use in vivo.

- Curing lights with power density greater than 800 mW/cm². Cure the buccal and lingual with 10 second exposures for each area.
- Curing lights with power density less than 800 mW/cm². Cure the buccal and lingual for 20 second exposures for each area.
- Curing lights with power density less than 300 mW/cm² should not be used to cure.

STORAGE

Refrigerate Infinity SE Syringeable for maximum shelf life.

Do not expose to temperatures exceeding 77° F (25° C).

Do not expose to direct sunlight.

Do not freeze.

Bring to room temperature before use.

RELATED PRODUCTS AVAILABLE FROM DEN-MAT

Description	Kit Number
Infinity SE Syringeable Kit.....	039680200
Infinity SE Syringeable Value Kit.....	039681200
Sapphire Plus Plasma Arc Curing Light.....	033968000
Flashlite Magna 4.0 LED Curing Light.....	CR1079
Tenure Multi-Purpose Bonding System.....	031146000
Auto-mixing tips/10 pc	030795525

SAFETY DATA SHEETS AVAILABLE AT denmat.com

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