## DOCTOR INSTRUCTIONS

### 1. Oral Exam

Examine the general condition of teeth, soft tissue health, and record the current tooth shade.

### 2. Mouthguard Fabrication

Follow the instructions below about mouthguard fabrication. Manufacture mouthguard in your office or send an impression to the **DenMat® Lab**.

### 3. Instructions to Patient (Review the "Instructions to the patient" with your patient):

#### Formula 16%

The recommended bleaching procedure is 4-8 hours a day for 14 days (overnight), or until desired results are achieved.

#### Formula 22%

The recommended bleaching procedure is 1-2 hours a day for 14 days (max. twice daily), or until desired results are achieved.

#### Formula 32%

The recommended bleaching procedure is 10-15 minutes a day, twice a day, for 14 days.

### 4. Recall Appointment

Advise the patient that whitening is usually stable for many months. However, certain foods, beverages and tobacco products may cause gradual restaining. In these cases, a maintenance program may be necessary, such as touch-up whitening for 1–2 days as needed.

Schedule a recall appointment in 3 weeks.

### Recommended Procedure for Fabricating a LUMIBRITE Whitening Mouthguard

1. Take an impression of the arch(es) you want to treat. The impression should allow all areas of the clinical crown and at least 1/2 inch of gingival tissue surrounding the teeth to be accurately represented.
2. Immediately pour a fast-setting stone mixture into the impression.
3. After the model has set, trim the base to make it as thin as possible (1/16"–1/8"). A hole carefully drilled in the center of the base will improve the vacuum-forming process.
4. Apply a thin layer of **Paint-On Dental Dam** onto the labial and lingual surfaces of the stone model to create space for the whitening gel. Light cure the **Paint-On Dental Dam**. This may also be accomplished by using traditional die space.
5. Place a 5"x 5" sheet of .040 or .080 clear resin mouthguard material into the clear resin holder on the upper part of the vacuum former.
6. Set the stone model in the center of the perforated plate of the former and allow the resin sheet to heat until it droops 1" from the clear resin holder. Activate the vacuum switch and lower the resin sheet onto the model.
7. When the mouthguard resin has cooled, remove it and the model from the vacuum former. Remove the resin from the model and trim (scallop) the labial and lingual surfaces of the mouthguard to the gingival margins.
8. Polish the scalloped margins on a lathe using a cloth wheel only. Edge transparency can be recovered by the careful use of an alcohol torch. Wash and clean the mouthguard thoroughly.

### Supplemental Products:

LUMIBRITE Professional Home Whitening Kit 16% Fruit	034369145
LUMIBRITE Professional Home Whitening Kit 16% Mint	034359145
LUMIBRITE Professional Home Whitening Kit 22% Fruit	034349145
LUMIBRITE Professional Home Whitening Kit 22% Mint	034339145
LUMIBRITE Professional Home Whitening Kit 32% Fruit	034239145
LUMIBRITE Professional Home Whitening Kit 32% Mint	034139145
LUMIBRITE Professional Home Whitening Kit 16% Fruit Refill	034368725
LUMIBRITE Professional Home Whitening Kit 16% Mint Refill	034358725
LUMIBRITE Professional Home Whitening Kit 22% Fruit Refill	034348725
LUMIBRITE Professional Home Whitening Kit 22% Mint Refill	034338725
LUMIBRITE Professional Home Whitening Kit 32% Fruit Refill	034238725
LUMIBRITE Professional Home Whitening Kit 32% Mint Refill	034138725

### Warnings:

- Keep out of reach of children.
- LUMIBRITE Professional Home Whitening System is supplied in a pre-filled syringe. This material is not intended for injection.
- LUMIBRITE Professional Home Whitening System is not intended for patients who are pregnant or nursing.
- LUMIBRITE Professional Home Whitening System should not be used by anyone with known allergy or sensitivity to peroxides.
- Patients with exposed dentin should be assessed for suitability to whitening prior to being given this product.
- Patient expectations should be discussed before treatment. Some teeth may not attain desired results such as existing crowns and bridges, and heavily restored teeth.
- Esthetic restorative procedures should be delayed until at least two weeks after whitening to ensure a good color match.

### Questions?

Contact your account representative at **800-433-6628**.

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LUMIBRITE Professional Home Whitening Kit 16% Fruit.....	034369145
LUMIBRITE Professional Home Whitening Kit 16% Menthe.....	034359145
LUMIBRITE Professional Home Whitening Kit 22% Fruit.....	034349145
LUMIBRITE Professional Home Whitening Kit 22% Menthe.....	034339145
LUMIBRITE Professional Home Whitening Kit 32% Fruit.....	034239145
LUMIBRITE Professional Home Whitening Kit 32% Menthe.....	034139145
LUMIBRITE Professional Home Whitening Kit 16% Fruit (recharge).....	034368725
LUMIBRITE Professional Home Whitening Kit 16% Menthe (recharge).....	034358725
LUMIBRITE Professional Home Whitening Kit 22% Fruit (recharge).....	034348725
LUMIBRITE Professional Home Whitening Kit 22% Menthe (recharge).....	034338725
LUMIBRITE Professional Home Whitening Kit 32% Fruit (recharge).....	034238725
LUMIBRITE Professional Home Whitening Kit 32% Menthe (recharge).....	034138725

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