LUMIBRITE®

chairside whitening

LUMIBRITE Chairside Whitening is designed to comfortably and quickly whiten patients' teeth. The combination of LUMIBRITE desensitizing enhancer and LUMIBRITE whitening gel reduces chairtime required for effective whitening compared to traditional chairside bleaching treatments. This kit contains enough material to whiten the upper and lower arches of one patient.

The LUMIBRITE Chairside Whitening kit is for in-office use only.

STORAGE: Store the LUMIBRITE whitening gel syringe in the refrigerator.

WARNINGS:

- Product contains both strong oxidizing and alkaline agents and can cause chemical burns. Wear protective gloves and eyewear during preparation and application.
- Avoid contacting soft gingival tissue with both the desensitizing enhancer and whitening gel. Using the Paint-On Dental Dam included in the kit will shield gingival tissue from these materials. Vacuum LUMIBRITE whitening gel off teeth as instructed before rinsing with water to prevent splattering. Should gel come in contact with soft tissues, rinse thoroughly to dilute.
- Material is not for injection.
- Keep out of the reach of children.

INSTRUCTIONS FOR USE:

Remove the LUMIBRITE whitening gel syringe from the refrigerator and allow it to come to room temperature for about 30 minutes before it is used.

1. Prophylax teeth to remove calculus and extrinsic stains before beginning the whitening treatment.
 2. Take "before" photographs and record tooth color using a shade guide arranged in bleaching order.
 3. Insert cheek retractors. Have the patient bite down and rest tongue on a tongue block.
 4. Air-dry the gingival tissue and teeth.
 5. Using the dispensing tip, apply Paint-On Dental Dam to the gingiva to isolate gums around the selected teeth. Overlap the dental dam onto the gingival tissue of adjacent teeth (about 0.5 mm) and light-cure. For curing directions, follow device instructions.
 6. Once the Paint-On Dental Dam is cured, apply Lip Moisturizer provided in the kit to the lips and mucosa, applying well beyond the vermilion border.
 7. Remove the tip from the end of the LUMIBRITE desensitizing enhancer recappable syringe and dispense a portion of the LUMIBRITE desensitizing enhancer into a dappen dish. Apply a very thin (about 0.2 mm) layer on the labial surface of the teeth using the brush applicator provided in the kit and allow to sit for one minute. Wipe off any excess. Firmly recap the LUMIBRITE desensitizing enhancer syringe.
 8. Prepare the auto-mix syringe.
 - Align the wedge-shaped key on edge of the static mixer housing with the opening in the syringe flange.
 - Push the static mixing tip onto the syringe and turn 90° clockwise until it stops.
 9. Dispense a portion of the LUMIBRITE whitening gel into separate dappen dish. Use the brush applicator to apply a 1–1.5 mm layer onto the labial surface of the teeth. Distribute the gel evenly over the teeth treated in Step 7 with the LUMIBRITE desensitizing enhancer. Agitate to mix LUMIBRITE desensitizer and LUMIBRITE whitening gel.
 10. Set the timer to 30 minutes. Once 15 minutes has elapsed, check on the comfort of the patient.
 11. Once 30 minutes has elapsed, leave the Paint-On Dental Dam in place and vacuum gel from the teeth. Wipe with gauze and rinse the teeth while suctioning. Avoid splatter. Some patients with heavy discoloration or areas of unseen hypocalcification may need another 30-minute session to achieve desired whitening results. Evaluate the results with the patient, if possible.
- Note:** In some cases you may have attained acceptable whitening in the first 30-minute session. If so, a second session is not indicated.
- Note:** In some cases, it may be necessary to peel away the Paint-On Dental Dam to make a shade determination. If the Paint-On Dental Dam is removed and another 30-minute application is indicated, remember to reapply the Paint-On Dental Dam to protect the gingival tissue.
12. If another 30-minute session is desired, reapply both desensitizing enhancer and whitening gel following instructions beginning with Step 7.
- Note:** Be sure to inspect the Paint-On Dental Dam for any cracking or lifting before re-applying the desensitizing enhancer and whitening gel. If the seal has broken from the tissue, it is recommended to remove and replace the Paint-On Dental Dam and begin with Step 4.
13. At the end of the 30-minute session, vacuum gel from the teeth, wipe away with gauze and rinse the teeth while suctioning. Avoid splatter.
 14. Use a shade guide arranged in bleaching order to compare the color of the whitened teeth to the original shade recorded in Step 2 and take "after" photographs.

STEP 2

STEP 5

STEP 7

STEP 9

STEP 12

STEP 13

STORAGE

Store the LUMIBRITE whitening gel syringes under refrigeration. Bring to room temperature before use.

SUPPLEMENTAL PRODUCTS

LUMIBRITE Take-Home Whitening Kit 16% Fruit
 LUMIBRITE Take-Home Whitening Kit 16% Mint
 LUMIBRITE Take-Home Whitening Kit 22% Fruit
 LUMIBRITE Take-Home Whitening Kit 22% Mint
 LUMIBRITE Take-Home Whitening Kit 32% Fruit
 LUMIBRITE Take-Home Whitening Kit 32% Mint
 LUMIBRITE Take-Home Whitening Kit 16% Fruit Refill
 LUMIBRITE Take-Home Whitening Kit 16% Mint Refill
 LUMIBRITE Take-Home Whitening Kit 22% Fruit Refill
 LUMIBRITE Take-Home Whitening Kit 22% Mint Refill
 LUMIBRITE Take-Home Whitening Kit 32% Fruit Refill
 LUMIBRITE Take-Home Whitening Kit 32% Mint Refill

ITEM NUMBER

034369145
 034359145
 034349145
 034339145
 034239145
 034139145
 034368725
 034358725
 034348725
 034338725
 034238725
 034138725

SAFETY DATA SHEETS ARE AVAILABLE AT denmat.com

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