



## CeriSander™ Posterior Finishing System



REF 031347500

### CONTENTS:

- CeriSaw™ Handles without blade, 4 pc. CE
- CeriSander Posterior Short Strips, fine, 6 pc. CE
- CeriSander Posterior Short Strips, ultra-fine, 6 pc. CE
- Instruction Card (NM-D)



MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany



Symbols Glossary available at:  
[denmat.com/symbols](http://denmat.com/symbols)

Den-Mat Holdings, LLC  
1017 W. Central Avenue  
Lompoc, CA 93436 USA

©2018 Den-Mat Holdings, LLC. All rights reserved.  
753475050 R-4 11/18CM

The CeriSander™ is designed to smooth and finish interproximal surfaces. Comes in both anterior and posterior versions. Sanding strips (fine and ultra-fine grit) are provided with the CeriSander. The CeriSander is used in conjunction with the CeriSaw™. The CeriSander and CeriSaw both utilize a common handle that can function as a saw (surgical blades) and as a sander (sanding strips).

The CeriSander Finishing System is provided mechanically clean but is not sterile. It should therefore be sterilized before use. Autoclave both the handle and strips for a holding time not less than 3 minutes at a temperature of between 134 and 137°C. Other validated times/temperature regimens may be used. The holding time is the minimum time for which the minimum temperature is sustained.

## DIRECTIONS

1. Place a wooden wedge interproximally.
2. Use the CeriSaw to separate the teeth. Leave the wooden wedge in place.
3. Use the CeriSander (fine grit) to smooth the interproximal surfaces.
4. Remove the wooden wedge.
5. Use the CeriSander (ultra-fine grit) to further smooth the interproximal surfaces. It is important that you use each sanding blade only two or three times to prevent open contacts.
6. Check the interproximal surfaces with dental floss for smoothness.



## Adding a strip:

1. Loosen the handle grip by turning handle grip counter-clockwise.
2. Attach CeriSander strip to the pins located at the ends of the handle. (Fig. 1)
3. Turn the handle grip clockwise to tighten the handgrip, which also tightens the pins and CeriSander strip. (Fig. 2)

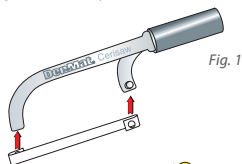


Fig. 1

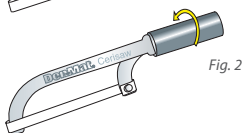


Fig. 2

## Removing a strip:

1. Loosen the handle grip by turning the handle grip counter-clockwise.
2. Lift the sanding strip from the pins.

## CERISANDER AND RELATED PRODUCTS

Description	Kit Number
Lumineers® Finishing Kit .....	033897100
CeriSaw Anterior Handle w/1 blade.....	031336510
CeriSaw Anterior Handle w/11 blades .....	031336500
CeriSaw Anterior Replacement Blades (10 pc.).....	031336520
CeriSaw Posterior Handle w/1 blade .....	031336610
CeriSaw Posterior Handle w/11 blades.....	031336600
CeriSaw Posterior Replacement Blades (10 pc.) .....	13371
CeriSander Anterior Kit w/strips .....	031337500
CeriSander Anterior Fine Strips (6 pc.) .....	031337510
CeriSander Anterior Ultra-Fine Strips (6 pc.).....	031337520
CeriSander Posterior Kit w/strips.....	031347500
CeriSander Posterior Fine Strips (6 pc.).....	031347510
CeriSander Posterior Ultra-Fine Strips (6 pc.) .....	031347520