



## PorceLock® Syringe Porcelain Etching Solution



REF 402060

### CONTENTS:


- PorceLock Porcelain Etching Solution, 3 pc. @ 1.75 ml ea. CE
- Syringe Tip, 18 G, 6 pc. (NM-D)
- Instruction Card (NM-D)

**For porcelain only.**  
**DO NOT contact gingival tissue.**  
**Store in a cool, dry location.**



MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany

Symbols Glossary available at:  
[denmat.com/symbols](http://denmat.com/symbols)

 Den-Mat Holdings, LLC  
1017 W. Central Avenue  
Lompoc, CA 93436 USA

©2019 Den-Mat Holdings, LLC. All rights reserved.  
754020690 R8.06/19CM

PorceLock® Porcelain Etching Solution is a buffered hydrofluoric acid, porcelain etching solution that makes exceptionally strong bonds to porcelain possible. Localized tissue protection can be established with Paint-On Dental Dam (No. 404210). In cases where bonding to metal and porcelain is required, we recommend Tenure® Multi-Purpose Bonding System (No. 031146000). For optimal bond strength to metal, use GoldLink® 2 (No. 13240) to bond to metal and mask dark metal shading.

**CAUTION:** Wear protective gloves while using this product.

**CAUTION:** Wear eye protection while using this product.

**CAUTION:** PorceLock has not been studied in children, pregnant or breast-feeding women.

#### DIRECTIONS FOR USE

1. Clean area thoroughly. Using a rough diamond bur, remove any related loose or friable porcelain. Remove surface glaze from the porcelain areas you will be bonding to.
2. Isolate the area by using Paint-On Dental Dam, or a rubber dam, on the soft tissue adjacent to the porcelain being etched.
3. Place 1-3 drops of PorceLock Porcelain Etching Solution in a dappen dish. Apply to the porcelain surface with either a disposable brush or Skubes.® Leave in place for a minimum of 3 minutes, making sure the surface remains moist. Thoroughly wash for 30 seconds. Peel away the Paint-On Dental Dam with an explorer and continue with the bonding procedure. To ensure strong bonds to porcelain, always use Cerinate Prime® and Porcelain Conditioner (No. 1399). In cases involving exposed metal, always use Tenure (No. 031146000) and GoldLink 2 (No. 13240).

#### CAUTION

PorceLock contains a weak, buffered solution of hydrofluoric acid. Avoid contact with skin, eyes, and soft tissue. If contact occurs, immediately rinse off with large amounts of water.

#### STORAGE

**Do not** expose to temperatures exceeding 77° F (25° C).

**Do not** expose to direct sunlight.

**Do not** freeze.

Cold storage is not required.

#### RELATED PRODUCTS AVAILABLE FROM DEN-MAT:

Description	Product Number
Paint-On Dental Dam Kit.....	404210
GoldLink 2 Kit .....	13240
Cerinate Prime.....	1398B
Porcelain Conditioner .....	1002AB
Tenure Multi-Purpose Bonding System.....	031146000
PorceLock Porcelain Etching Solution Kit.....	402060
Skubes 900 pc.....	0065
Sapphire® Plus Plasma Arc Curing Light.....	033958000

#### SAFETY DATA SHEETS AVAILABLE AT [denmat.com](http://denmat.com)