



Desensitizing Kit Activation Enhancer for Chairside Whitening



REF 034159555

CONTENTS:

- Lumibrite Desensitizing Gel syringe, 3 pc. @ 1 ml ea.
- Instruction Card



Symbols Glossary available at:
denmat.com/symbols



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

©2019 Den-Mat Holdings, LLC. All rights reserved.
754158550 R.3 03/19CM

Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA

Lumibrite Professional Chairside Whitening is designed to comfortably and quickly whiten patients' teeth. The combination of Lumibrite Activation Enhancer and Lumibrite Whitening Gel reduces the chairtime required for effective whitening compared to traditional chairside bleaching treatments.

The Lumibrite Professional Whitening Kit is for in-office use only.

- The Activation Enhancer contains strong alkaline agents and can cause irritation to unprotected skin or gingiva. Wear protective gloves and eyewear during preparation and application.
- Avoid contacting soft gingival tissue with the Activation Enhancer. Using the Paint-On Dental Dam included in the Lumibrite Professional Whitening Kit will shield gingival tissue from this material.
- Material is not for injection.
- Keep out of the reach of children.
- Follow all instructions and cautions for the Lumibrite Whitening System found in the Lumibrite Professional Chairside Whitening Kit.
- Lumibrite Activation Enhancer is intended for use ONLY with the Lumibrite Whitening Gel.

INSTRUCTIONS FOR USE:

Remove the Lumibrite Whitening Gel syringe from the refrigerator and allow it to come to room temperature for about 30 minutes before it is used.

1. Prophyl teeth to remove calculus and extrinsic stains before beginning the whitening treatment.
2. Take "before" photographs and record tooth color using a shade guide arranged in bleaching order.
3. Insert cheek retractors. Have the patient bite down and rest tongue on a tongue block.
4. Air-dry the gingival tissue and teeth.
5. Using the dispensing tip, apply Paint-On Dental Dam to the gingiva to isolate gums around the selected teeth. Overlap the dental dam onto the gingival tissue of adjacent teeth about 0.5 mm and light-cure. For curing directions, follow device instructions.
6. Once the Paint-On Dental Dam is cured, apply Lip Moisturizer provided in the kit to the lips and mucosa, applying well beyond the vermilion border.
7. Remove the tip from the end of the Lumibrite Activation Enhancer recappable syringe and dispense the Lumibrite Activation Enhancer in a very thin (about 0.2 mm) layer on the labial surface of the teeth. Distribute the gel evenly using the brush applicator provided in the kit and allow to sit for one minute. Wipe off any excess. Firmly recap the Lumibrite Activation Enhancer syringe.
8. Prepare the auto-mix syringe.
 - Align the wedge-shaped key on the edge of the static mixer housing with the opening in the syringe flange.
 - Push the static mixing tip onto the syringe and turn 90° clockwise until it stops.
9. Dispense a portion of the Lumibrite Whitening Gel into a separate dappen dish. Use the brush applicator to apply a 1 – 1½ mm layer onto the labial surface of the teeth. Distribute the gel evenly over the teeth treated in Step 7 with the Lumibrite Activation Enhancer. Agitate to mix the Lumibrite Activation Enhancer and Lumibrite Whitening Gel.

10. Set the timer to 30 minutes. Once 15 minutes has elapsed, check on the comfort of the patient.
11. Once 30 minutes has elapsed, leave the Paint-On Dental Dam in place and vacuum gel from the teeth. Wipe with gauze and rinse the teeth while suctioning. Avoid splatter. Some patients with heavy discoloration or areas of unseen hypocalcification may need another 30-minute session to achieve desired whitening results. Evaluate the results with the patient, if possible.

Note: In some cases you may have attained acceptable whitening in the first 30-minute session. If so, a second session is not indicated.

Note: In some cases, it may be necessary to peel away the Paint-On Dental Dam to make a shade determination. If the Paint-On Dental Dam is removed and another 30-minute application is indicated, remember to reapply the Paint-On Dental Dam to protect the gingival tissue.

12. If another 30-minute session is desired, reapply both Activation Enhancer and Whitening Gel following instructions beginning with Step 7.

Note: Be sure to inspect the Paint-On Dental Dam for any cracking or lifting before re-applying the Activation Enhancer and Whitening Gel. If the seal has broken from the tissue, it is recommended to remove and replace the Paint-On Dental Dam and begin with Step 4.

13. At the end of the second 30-minute session, vacuum gel from the teeth, wipe away with gauze and rinse the teeth while suctioning. Avoid splatter.
14. Use a shade guide arranged in bleaching order to compare the color of the whitened teeth to the original shade recorded in Step 2 and take "after" photographs.

SAFETY DATA SHEETS AVAILABLE AT denmat.com

LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

DENMAT CUSTOMER SUPPORT

U.S. and Canada Customer Service.....	800-433-6628
U.S. and Canada Fax Orders and Inquiries.....	800-233-6628
International Inquiries	805-347-7990
International Fax Orders and Inquiries:	805-347-7983
Website	denmat.com