



varnish with Xylitol
5% sodium fluoride



instructions for use

Pro-DenRx 5% Sodium Fluoride Clear Varnish Instructions for Use

DESCRIPTION: Pro-DenRx 5% Sodium Fluoride Clear Varnish is a fluoride varnish which has a strong desensitizing action when applied to dental surfaces. Pro-DenRx varnish may be used on moist teeth and leaves a film of varnish which sets in the mouth and may allow visual control and verification.

INDICATIONS: Pro-DenRx 5% Sodium Fluoride Clear Varnish is a fluoride-containing preparation for the treatment of dentin hypersensitivity, and for the reduction of post-operative sensitivity. 1ml Pro-DenRx fluoride varnish contains: 50 mg of sodium fluoride, equivalent to 22.6 mg fluoride ion.

DOSAGE: For the treatment of dentin hypersensitivity, two applications at several days interval are sufficient in most cases. The fluoride content of Pro-DenRx fluoride varnish is dosed in such a way that neither acute nor chronic side effects are to be expected if applied according to instructions

APPLICATION INSTRUCTIONS

1. Peel open the foil lidding material
2. Remove brush and mix varnish until it reaches an even consistency
3. For best results, dry teeth prior to application by rubbing a 2x2 (or other disposable cotton sponge) along the tooth surfaces to remove any existing biofilm
4. Apply a very thin coat of varnish evenly to all of the desired tooth surfaces
5. Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth
6. Varnish hardens on contact with saliva so the patient may leave immediately after application
7. When dry, any excess varnish may appear "spotty" and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth
8. It is recommended that the patient be instructed to only eat soft foods and drink cold liquids for two hours after treatment

CAUTION: Federal law restricts the use of this device to sale by or on the order of a medical or dental professional. For use in the oral cavity.

CONTRAINDICATIONS: Ulcerative gingivitis and stomatitis.

INTERACTIONS: When Pro-DenRx fluoride varnish is applied, other fluoride preparations such as fluoride gels should not be administered during the same day. The routine use of fluoride tablets should be interrupted for several days after treatment.

ADVERSE REACTIONS: Edematous swellings have been reported only in rare instances in some fluoride varnish products, especially after application to extensive surfaces. Dyspnea, although extremely rare, has occurred in asthmatic children. Nausea has been reported when extensive applications have been made. If required, varnish film can be removed with a thorough brushing.

SUPPLY: Individual unit-dose packages containing 0.5 ml sodium fluoride clear varnish.

REORDER INFORMATION:

Reorder Part #	Description
VN200BG	Pro-DenRx 5% NaF Varnish Clear Bubblegum (Box/200)
VN035BG	Pro-DenRx 5% NaF Varnish Clear Bubblegum (Box/35)
VN200ML	Pro-DenRx 5% NaF Varnish Clear Melon (Box/200)
VN035ML	Pro-DenRx 5% NaF Varnish Clear Melon (Box/35)
VN200MT	Pro-DenRx 5% NaF Varnish Clear Mint (Box/200)
VN035MT	Pro-DenRx 5% NaF Varnish Clear Mint (Box/35)



 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA

800-433-6628 | denmat.com