

# ub Ultra-Bond®

Try-In Paste for Resin Cement  
Refill Kit, BØ

dm  
DenMat®



## CONTENTS:

- Ultra-Bond Plus Try-In Paste Syringes, BØ, 4 pc. @ 1 g ea. **CE 0044**
- Tenure® S Dab-Eze® 3 g **CE 0044**
- Syringe Tips 18G, 12 pc. (NM-D)
- All-Surface Bonding Chart (NM-D)
- Instruction Card (NM-D)



R<sub>X</sub> ONLY



LOT



EC REP

MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany

Symbols Glossary available at:  
[denmat.com/symbols](http://denmat.com/symbols)

Den-Mat Holdings, LLC  
1017 W. Central Avenue  
Lompoc, CA 93436 USA

REF 031982210

©2019 Den-Mat Holdings, LLC. All rights reserved.  
750198120 R:4 05/19CM

**CAUTION:** Wear eye protection while using this product.

**CAUTION:** Wear protective gloves while using this product.

**CAUTION:** UltraBond products contain methacrylate monomers which have been known to cause allergic reaction in individuals sensitive to these ingredients.

**CAUTION:** Ultra-Bond has not been studied in children, Pregnant or Breast-feeding women.

## TRY-IN

### 1. Preparation of Lumineers®

- After the patient is seated, treat the Lumineers with Porcelain Conditioner for 30 seconds, rinse and dry well.
- Apply Cerinate Prime® with a disposable brush to the Lumineers for 30 seconds and blow thin. (DO NOT use cotton pellet or sponge.)
- Place Lumineers back into the case box in their proper slots.
- Close the lid on case box to shield from light.

### 2. Preparation of Tooth Surfaces

- Clean the teeth with a slurry of Porcelain Polishing Paste.
- Etch teeth for 20 seconds with Etch 'N' Seal,® rinse and dry.
- Mix together equal amounts of Tenure® A and B. Apply 3-5 coats (or until glossy) to each tooth, then air-dry the tooth surfaces.

### 3. Try-In of Lumineers

- Syringe the Ultra-Bond® Plus Try-In Paste into each of the Lumineers.

Note: If using light-cured TetraPaque® to modify Try-In Paste, protect Lumineers from exposure to intense light.

- Gently insert the Lumineers one at a time and use a brush to clean up the excess Try-In Paste.

### 4. Show Patient

- Hold a mirror approximately 16 inches away from the patient's face and let them examine their smile.

- If the shade is not what you or the patient wants, remove the Try-In Paste from the Lumineers and tooth surface with a Skubex® or a clean, dry brush. Reapply a new shade of Try-In Paste and check the shade again.

### 5. To Remove Lumineers After Try-In

- Use a Schure 349 instrument or cotton pliers and gently tease the Lumineers off the tooth surfaces.
- Remove the Try-In Paste from the Lumineers and tooth surfaces with a clean brush. DO NOT USE Tenure S. Only use Tenure S to clean up excess once final Try-In Paste has been selected.
- Gently thin the Tenure S with air. A thin film will be left on both the tooth and porcelain surfaces.

Note: There is no need to reapply Porcelain Conditioner or Cerinate Prime if you are cementing directly after try-in.

For application of Lumineers, see the Ultra-Bond Plus cement kit instructions.

## STORAGE

**Do not** expose to temperatures exceeding 75° F (24° C).

**Do not** freeze.

## RELATED PRODUCTS AVAILABLE FROM DENMAT

Description	Kit Number
Ultra-Bond Plus Multi-Shade.....	032599750
Tenure Uni-Bond® with Gloss-N-Seal®.....	030411900
Tenure MPB System Kit.....	031146000
Porcelain Conditioner.....	1002AB
Cerinate Prime.....	1398B
Tenure S Dab-Eze.....	031145201
Sapphire® Plus Plasma Arc Curing Light.....	033968000

**SAFETY DATA SHEETS AVAILABLE AT [denmat.com](http://denmat.com)**