




BondLink
Bond Enhancer
for Self Curing
Composite Restorations



CONTENTS:

- BondLink 6 ml **CE 0044**
- Dappen dish **(NM-D)**
- Brush tip, 5 pc. **(NM-D)**
- Brush tip handle **(NM-D)**

 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA

R_X ONLY



REF 031148100

EC	REP	MDSS GmbH Schiffgraben 41 30175 Hannover, Germany
----	-----	---

Symbols Glossary available at: www.denmat.com/symbols

©2018 Den-Mat Holdings, LLC. All rights reserved. 751148150 R:6 06/18CM

BondLink is a coupling agent that bonds between single-bottle adhesives and self-cure composites. BondLink is recommended for use with self/dual-cure composites when using any light cure bonding agent that you are unable to reach with a curing light in order to activate.

CAUTION: Wear protective gloves while using this product.

CAUTION: Wear eye protection while using this product.

CAUTION: Bondlink has not been studied in children, pregnant or breast-feeding women.

DIRECTIONS:

After applying a light cure bonding agent, apply one coat of BondLink to the prepared surface and gently air-dry. Proceed according to standard composite application instructions.

STORAGE:

Store at room temperature.

Do not expose to temperatures exceeding 75° F (24° C).

Do not expose to direct sunlight.

RELATED PRODUCTS AVAILABLE FROM DENMAT:

DESCRIPTION	PRODUCT NUMBER
Tenure® Multi-Purpose Bonding System.....	031146000
Tenure Quik® with Fluoride.....	031145700
Tenure® Uni-Bond®	030411900

LIMITED WARRANTY

DenMat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. DenMat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

DENMAT CUSTOMER SUPPORT

U.S. and Canada.....	800-433-6628
U.S. and Canada Fax Orders and Inquiries.....	800-233-6628
International Inquiries	805-347-7990
International Fax Orders and Inquiries:	805-347-7983
Website	www.denmat.com