GRIPPERTRAY®

Super Rigid Bite Trays

Directions For Use

Mode d'emploi

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Bruksanvisning

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The Gripper Tray®

Super Rigid Bite Trays

DESCRIPTION: Disposable super rigid, plastic tray frame with thin mesh webbing used to capture simultaneous crown and bridge, bite registrations and opposing impressions.

DIRECTIONS FOR USE:

1. Place unloaded Gripper Tray into proper position. When using the posterior quadrant tray, make sure the end of the tray is distal to the last maxillary tooth and the tray is centered buccally/lingually.
2. Instruct the patient to "close" on the unloaded tray. Inspect the opposite side of the patient's mouth to ensure that the patient is closed into the correct occlusion. To record the most accurate bite registration, take the impression with the patient in an upright position.
3. Paint impression adhesive on tray walls only, not on the webbing. Overload both sides of the tray.

NOTE: Gripper Tray must be used with a firm setting material. It may also be used with hard setting Heavy Body/Light Body one-step techniques. Two-step techniques and putty materials are not recommended.

4. Remove retraction cord if used, wash the area thoroughly. Syringe impression material around the area to be recorded.
 5. Load Gripper Tray with impression material.
 6. Place loaded Gripper Tray into proper position and have the patient close properly.
 7. When set, gently remove from mouth.
-

HOW SUPPLIED: 40 Gripper Trays per box.

SPD1414 Gripper Tray Anterior

SPD1417 Gripper Tray Full Arch Large

SPD1415 Gripper Tray Posterior

SPD1418 Gripper Tray Full Arch

SPD1416 Gripper Tray $\frac{3}{4}$ Arch

SPD1541 Gripper Tray Sideless Posterior

STORAGE: No particular storage requirements necessary.

PRECAUTIONS:

General: Single use only.

CAUTION: Federal (USA) Law restricts sale of this device to, by or on the order of a dental professional.

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Den-Mat Holdings, LLC
MDSS GmbH